


CareDry® System: Patient Placement Techniques


FEMALE ANATOMY

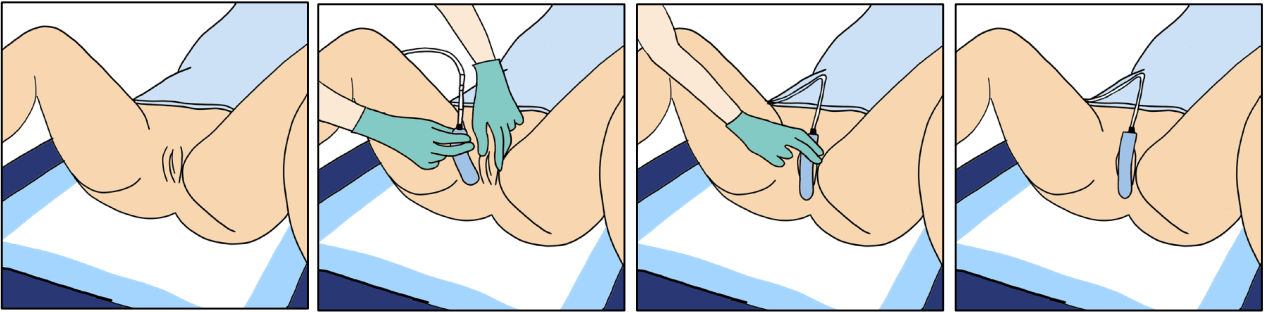

- Separate the patient's legs, gluteus, and labia.
- Tuck the device vertically, sponge down, between anatomy.
(Note: If needed, the CareDry® device can be bent to conform to the patient's anatomy.)


- Gently close the legs and place the tubing and connected side of the CareDry® device in a comfortable location where it is unlikely to become dislodged or crushed.
- Device is now ready for use.

EMACIATED - STATIONARY

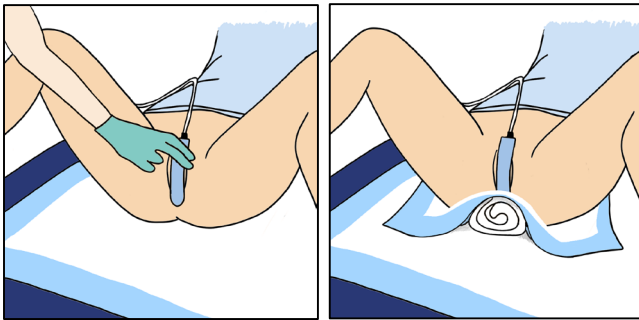


- Once placed based on the instructions above, fold a washcloth to the size of your closed fist and put it under a chuxs pad.
- Place the CareDry® device and edge the washcloth against chuxs to support the CareDry® sponge.

EMACIATED - RESTLESS

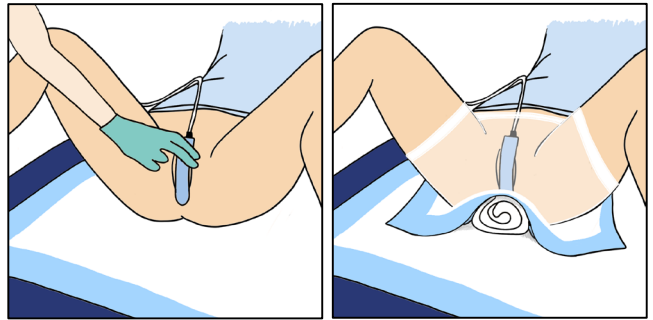

- Place CareDry® device based on instructions above and under "Emaciated –Stationary."
- Then use mesh panties to keep in position – You may use paper tape or stat lock to keep suction extension tubing in place.

OBESE ANATOMY

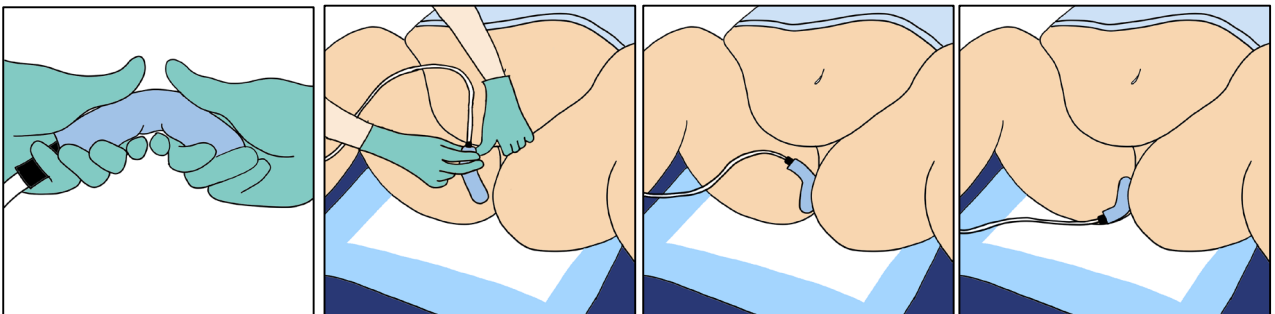

- Ensure the patient's flesh is supported for proper placement.
- Form the CareDry® device into an "S" shape to allow full engagement with the target urine evacuation zone.

- Place the CareDry® device with the connected tubing end either up or down.
- Ensure tubing is directed away from the patient's flesh to travel down the leg and over the bedside.


This guide provides suggestions as to how to best place the CareDry® in patients with different anatomies. Refer to the complete Instructions for use included with the product 7660 CareDry® system IFU 34393 for proper use of the device including indications, contraindications, and warnings.

BOEHRINGER

®


CareDry® System: Patient Placement Techniques

MALE ANATOMY

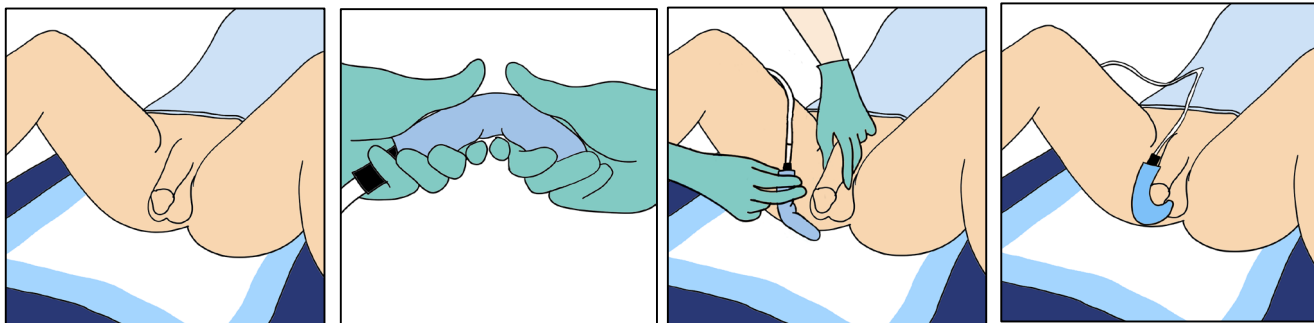


- Separate the patient's legs, bend the CareDry® device into a "U" shape, and place it around the tip of the penis.

3-OPTIONS

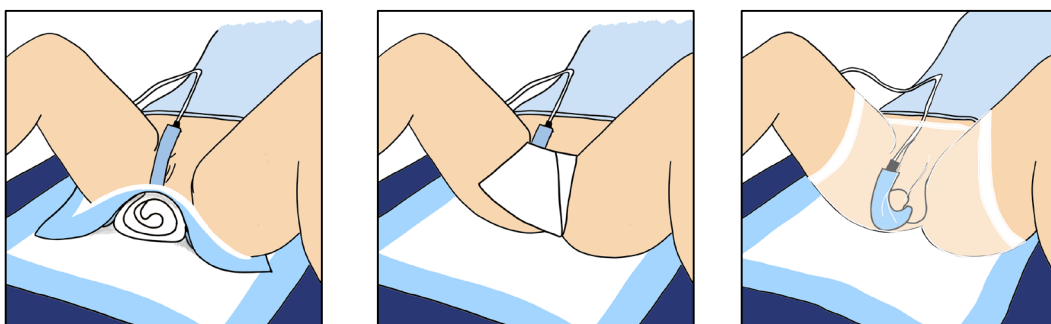

- Use a washcloth buttressed against chux, just chuxs, or a male incontinence pad to ensure maximum urine collection and dry patient.
- May use mesh brief and male incontinence pad for patients that move around in bed.
- Use a stat lock or paper tape to keep the suction extension in place.

BURIED PENIS

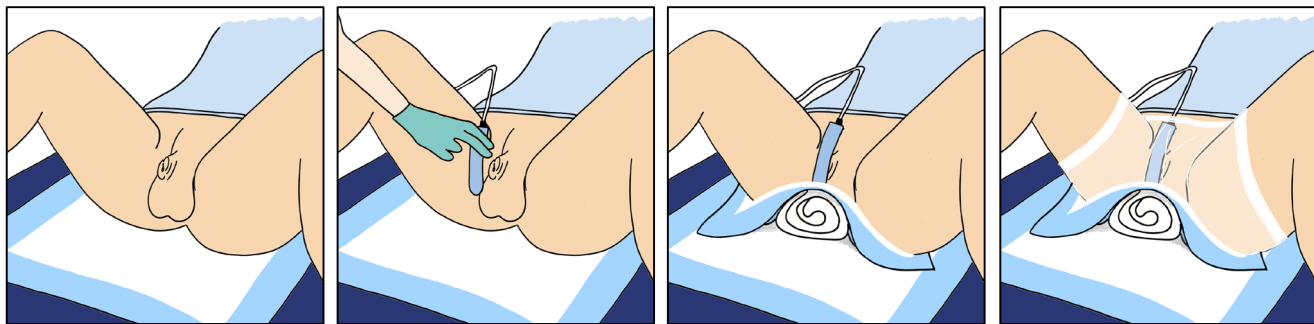

- Separate the patient's legs, bend the CareDry® device to conform to the slotted opening, or bend into a "U" shape and place it around the scrotum.
- Use a washcloth buttressed against chux, just chuxs, or a male incontinence pad to ensure maximum urine collection and dry patient.
- May use mesh brief and male incontinence pad for patients that move around in bed.
- Use a stat lock or paper tape to keep the suction extension in place.